**INFORMATION FOR USE** ..... (en)

=> **medical device: seating system**

**GEBRAUCHANWEISUNG** ..... (de)

=> **Medizinprodukt: Therapiestuhl**

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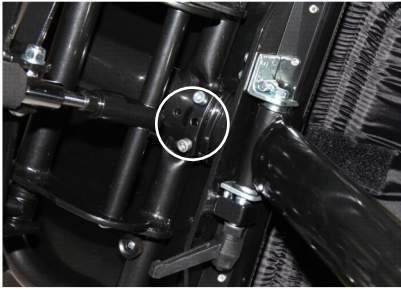
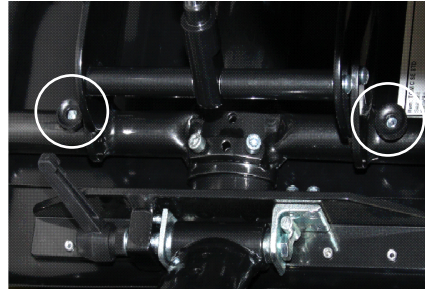
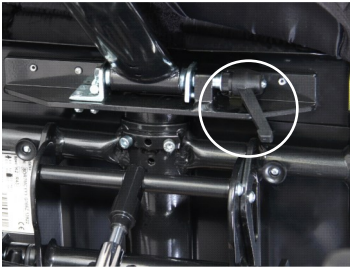
Pic. 20

**INTRODUCTION:**

Dear customers, read this document carefully before using this product for the first time. Follow all safety instructions stated in this document. If you have any questions about the product and its proper use, contact professional staff or a service centre. Report any serious adverse event related to the product, or any deterioration in the user's health in connection with the use of the product, immediately to the official distributor of the product in your country or to the manufacturer of the product. For latest safety and/or recall information for this product, as well as the product declaration of conformity, please contact the official distributor of the product in your country or the product manufacturer. Keep this document in a safe place for further reference.

..... **WARNING, CAUTION:**

.....**maximum load.**

**WARNING! Keep fingers out of any moving parts when operating with the stroller.**

**WARNING! Never leave your child unattended! Even if the belts and brakes are in use.**

**WARNING! Check before use that all locking devices are properly locked and all safety equipments are properly used.**

**WARNING! Check before use that seat unit is properly locked in the chassis interface.**

**WARNING! The seat unit is not suitable for a child younger 6 months.**

**WARNING! The seat unit must not be used in motor vehicles as the car seat.**

**ATTENTION! Read this manual carefully before the first use of the stroller, as well as keep it for further reference. Safety of a child might be in danger in case you will not follow the instructions and safety notices mentioned I the user guide.**

**PRODUCT - DESCRIPTION, FEATURES, IDENTIFICATION:**

You have received a product that is determined for versatile everyday usage as indoor at home. Before using the product for the first time, get to know carefully how to use it, how to handle with it, how works all the product features, to avoid any possibility of injury. In particular, observe the following instructions:

- All users and / or their accompanying persons must be trained by qualified personnel how to operate with the product. Users and / or their accompanying persons are obliged to read and learn in particular about the residual risks of this product, in connection with the safety instructions given in this document.
- The product has been adapted to the individual needs of the user. Additional changes may only be made by qualified personnel.
- We recommend that you check the product customization once a year to ensure optimal product setup for users. Especially for users with changing anatomy (eg body dimensions and weight) it is recommended to adjust at least once every six months.
- Your specific product may differ from the variants shown here. In particular, not all optional accessories described in this document are mounted on your product.
- The manufacturer reserves the right to make technical changes to the design described in this manual.

**Product features:**

This product is classified as a non-invasive therapeutic device of class 1. The product is intended for short-term and long-term use. The product is controlled by an accompanying person and is used to transport children with the help of another person. The product is intended for use on a solid surface in interiors.

The product is used by children with temporary or permanent limitation of the ability to walk, unable to walk or keep standing. The product is suitable for users whose anatomy (eg body dimensions and weight) allows the use of

this product for the given purpose and whose skin is intact. The product must only be used in conjunction with the accessories specified in the order form.

The manufacturer of this product assumes no responsibility for combinations with medical devices and / or accessories from other manufacturers. Only product combinations evaluated in terms of their efficiency and safety are permitted. This product and accessory combination must be permitted and stated in the product combination agreement.

Indications: mild to severe or complete restriction of the user's movement.

Contraindications: Absolute contraindications are not known. Relative contraindications - lack of physical or mental preconditions, significant surface damage of surface tissue (eg skin, etc.).

Accompanying phenomenon: Exceptionally, the following accompanying phenomena may occur when using the product - pain in the neck muscles, pain in the muscles and joints, circulatory disorders, risk of pressure sores. In case of these accompanying phenomena or problems, contact your doctor or therapist.

**Product identification:**

Information on the product label:

	Global trade identification number. According to GS1-standard = UDI-DI – Unique Device Identifier - Device Identifier. .
	Serial number. According to GS1-standard = UDI-PI – Unique Device Identifier - Production Identifier.
	Producer identification – product version
<b>Mod.:</b>	Product version / text
<b>Spec.:</b>	Product version / colour version
	Producer - address

Date of production: Serial number (first 3 digit) includes production date. („253“ = 253.-months from 01.01.2000 = January 2021)

**SAFETY AND PROTECTION:**

- The attendant must be familiar with the instructions of this manual.
- Professional medical advice regarding the suitability of this product must be sought before use.
- Always make sure, before departure, the child is comfortable, and that clothes and rugs etc, are tucked in, so they do not fall in the wheels.
- The stroller MUST NOT be used as a seat for a child while traveling in motor vehicle or bus.
- Ensure that child is clear of moving parts before making adjustments, to avoid any injury.
- When angle-positioning and adjusting the seat and backrest act accordingly to avoid sudden uncontrolled movements.
- Change of the seat direction must be done without the occupant and with empty seat unit.
- Do not allow the child to climb unassisted into and out from the stroller.
- Always apply the parking brakes, when getting in and out the stroller.
- The stroller is not designed to be driven down the stairs.
- Ensure yourself that the fixing accessories, safety bar and other equipment is fitted and adjusted correctly.
- Do not stand on the footrest.
- The brakes must be engaged at all times when the stroller is not moving.
- Getting over the curb practice firstly with an empty stroller.
- The attendant must take precautions against the fire risk. In buildings, check the location of fire exits and ensure procedures are understood.
- Do not let children play with or hang on the stroller.
- During the drive keep full attention, to avoid any dangerous situation or collisions.
- Do not overload the stroller. Overloading of a chassis, seat, bag or basket causes reduction of stability of the stroller.
- Make the full attention, when manipulate with the stroller, not to injure the child or yourself.
- If it is possible, always use drive-up ramps or lift to overcome any access restrictions.

- Always lift the stroller in two persons.
- Never transport the stroller when you and/or assistance are not sure if you can transport the stroller with the child safely.
- When you drive the stroller on the road, follow the road traffic regulations.
- Do not use the stroller for any other purposes than it is designed for.
- The stroller is designed to carry only one child.
- Do not use any other than original PATRON accessories designed for this product.
- When the stroller is folded and unfolded, the user guide must be followed carefully, to avoid any damage on the construction what could affect the product stability.
- In case of apparent defect, which limits product's safe using (e.g. damage of chassis, part falls away or other optically visible defects, cracks etc.) it is necessary to stop using the product immediately and contact your local official distributor or service centre.
- Company PATRON does not bear any responsibility for the damages caused by using other than original PATRON accessories supplied by this company.
- Company PATRON does not bear any responsibility for the damages and accidents caused by inappropriate use of this product.

## **PREPARATION FOR USE, USE:**

For assembly positioning products FROGGO you need allen wrench 4 mm. You will also need a flat wrench 10mm, this wrenches is not included.

To set positioning products FROGGO use the allen wrench , which can be found in the pocket equipped with Velcro on the back seat.

### **Seat:**

#### **Seat appointment –removal**

Appointment: brake the chassis . Place the seat on the chassis so that the holes in the seat frame fit exactly into the corresponding holes in the chassis . (Fig. 1 ) Srew the seat securely to the chassis. (Fig. 2 ) . By pulling the seat up , always check that it is properly secured ! Make sure the coxal rests, side frame protection and armrests covers are fitted properly.

Removal : brake the chassis . Unscrew the seat from the chassis . Release the seat and remove it .

#### **Adjusting the angle of the seat**

Slide under the pedal, turn and then again push the safety catch . Hold in hands the seatback. By depressing the left pedal you trigger a mechanism for tilting seat and you can set it to the desired angle. Then release the pedal again. (Fig. 3 )

#### **Adjusting the height of the seat**

Slide under the pedal, turn and then again push the safety catch . Hold in hands the seatback. By depressing the right pedal you release mechanism for height adjustment of the seat. Set the seat to the desired height and release the pedal again. (Fig. 4 )

**WARNING !** The ram lifting the seat is designed to lift the weight up to 50 kg. If you use the pedal height adjustment without load , it will very quickly raise the seat. There is risk of injury or damage of lifting equipment in this case.

#### **Adjusting the depth of the seat :**

Release the cover of seat by lifting and releasing the Velcro. Use allen wrench for unloosing two screws on the underside of the seat. (Fig. 5 ) Unloosen also four plastic nuts on the underside of the seat. (Fig. 6) Set the required depth of the seat. Then tighten the screws (allen wrench ) and also plastic nut (by hand ) . Thread the back seat cover .

#### **Set the width of the seat**

Release the seat cover from the sides of seat ( Velcro ) . Unloosen the four plastic nuts on the underside of the seat. (Fig. 6 ) Slide the side panels to set the required width of the seat. Tighten side plate by with the four nuts. Thread the back seat cover .

#### **Adjusting the angle of backrest :**

Unloosen the lever on the rear of the backrest . (Fig. 7 ) Adjust the backrest to the desired position and secure it by tightening the lever . Or slightly Unzip the cover of backrest support to facilitate handling . Check that the backrest is correctly locked into position. Backrest should not move in angle.

#### **Setting the height of the backrest :**

Loosen the cover of backrest , both on the back side ( Velcro ) and front ( Velcro ) . With allen wrench unloosen the two screws (Fig. 8) and manually unloosen the 2 plastic nuts on the rear of the backrest . (Figure 9 ) By pulling or inserting you can adapt the desired height of the backrest . Then tighten the screws (allen wrench ) and also plastic nut (by hand ) . Thread the backrest cover back . (Fig. 10)

## **Adjustability of components :**

### **Coxal rest:**

Attachment - Attach the sides of the coxal rest with plastic sleeves on the side of the vertical pipes of backrest .

(Fig. 11)

Removal - head coxal rest toward to backrest cover, remove the plastic sleeve from tubes

### **Armrests:**

The armrests are adjustable. You can set the height of the armrests by unloosen of screws , removing / pulling armrests and then tightening the screws with allen wrench. Angle of the armrests can be set by unloosen lever and pushing on one side of armrest . The correct angular position of the arms is signalled by click sound . After entering the correct angle of the armrests , be sure to tighten the lever . You can set the depth of the armrests by unloosen of lever then sliding armrests into on of three holes and tighten of lever. ( By MINI version you can only set the height of the armrests ) . For fixation of armrests attach the armrests cover and close Velcro .

### **Frontbar :**

To remove the Crossbar press two metal locking pins of frontbar inside the tubes. Slide the frontbar upwards. When attaching the locking pins must face forward . Push the frontbar to the respective tubes until it clicks into the locking pins . (Fig. 12) Pull the frontbar upwards and make sure it is properly fastened .

### **Adjusting the angle of the footrest :**

Loosen the lever on the underside of the seat , adjust the angle of the footrest , secure the position by tightening the lever . (Fig. 13)

### **Set the length of footrest :**

Use allen wrench , unloosen two screws on the back side of the tube of footrest . (Fig. 14) Pull out or insert footrest as needed to adjust the length. Then tighten the screws (allen wrench).

### **Adjusting the angle of the footrest :**

Use allen wrench , unloosen two screws on the underside of the footrest . Move the rectangular plate so that the blade when it is folded stops at the desired angle . (Fig. 15 ) Then tighten the screws (allen wrench).

### **Velcro leg support :**

With Velcro located on the underside of the footrest fix the position of the footrest to prevent accidental folding . (Fig. 16)

### **Cover :**

Stroller cover is removable and can be washed by hand (max. 40 ° C).

### **Cover of backrest:**

Thread above the rest , the lower part of the cover thread the gap between the backrest and the seat cushion and secure it with the Velcro on both the front and the rear of the backrest . (Fig. 17)

### **Seat Cover:**

Place on seat, press the Velcro . (Fig. 18)

### **Cover sidewalls:**

Thread on the front side of sidewalls , so that the color is coating always inside the seat. (Fig. 19)

### **Optional accessories :**

The product is offered with a wide range of special fixation and defining accessories. You can find manual for these accessories in separate brochure.

## **positioning products FROGGO handling :**

### **The parking brake**

Braking : after a full stop you can brake by pressing levers engage to the chassis . To check if chassis is properly braked always convince slight movement forward or reverse. (Fig. 20)

Releasing : lift lever brakes. The chassis is released.

### **Annealing :**

Brake the chassis . To facilitate the annealing is appropriate to ask a trusted person to hold the chassis in order . Carefully sit the child into the seat , fix all fixing accessories with which your positioning products FROGGO in compliance with the instructions below can be used for a long time without having to use the services of your service center.

Positioning products FROGGO is equipped,attach the frontbar and adjust the footrest to the desired position. Make sure that the child is always sitting comfortably .

### **Getting out:**

Brake the chassis . To facilitate getting out you should fold footrest and ask a trusted person to hold the chassis .

Unzip all fixing accessories which your positioning products FROGGO is equipped with, remove the frontbar and carefully remove the child from the stroller.

**WARNING !** Never lift the stroller by holding only frontbar, footrest or seat !

## **MAINTENANCE, PRODUCT DISPOSAL:**

Your pediatric positioning seat has been designed to the high standard and with correct usage and maintenance will provide many years of trouble free service.

- make sure that you are familiar with the operation of the stroller. It should always open and fold easily. If it does not, do not force the mechanism - stop and read the instructions
- regularly inspect safety bar fittings, locking devices, catches, footrest, joints fixings to ensure they are secure, clean and in full working order
- brakes, wheels, tyres and upholstery are subjects to wear and should be replaced if necessary by a service centre.
- if the stroller is damaged or in need of any repair, only authorised service centres are allowed to provided this services.
- only spare parts supplied by company PATRON has to be used for repairs.
- should parts of your stroller become stiff or difficult to operate, ensure that dirt is not the cause, apply a light application of a spray lubricant e. g. WD 40 (do not use oil or grease). Ensure that the lubricant penetrates between the components but do not overspray as this may encourage the accumulation of dirt. Wipe off any spray excess. Regular lubrication of moving parts will extend the life of your stroller and make opening and folding easier.
- do not expose the stroller to the long-acting intensive solar radiation
- textile components of stroller and/or accessory must be used and stored in the way that the transparent plastic parts are not folded or turned tightly. Those can be damaged if the temperature is 5°C below zero.
- wheels, plastic and metal parts may be sponged clean with warm water and mild detergent.
- the seat fabric must be hand washed only at maximum temperature of 40°C, do not use laundry dryer.
- never clean with abrasive, ammonia based, bleaches and alcohol based cleaning compounds.
- after exposure to damp conditions, dry the product off with a soft cloth and leave fully open in warm and dry environment.
- store your stroller in a dry, safe place with recommended temperature 5-20°C.
- it is recommended that the stroller is checked by an approved service centre every 12 month
- We recommend that after 24 months visit a service department.
- Check and tighten the screw connections regularly.
- If there are used control cables (brakes, telescopes, tilt), check it regularly and, if necessary, set up and oil them.

**Product lifetime:** The expected product lifetime is 5 years. The expected product lifetime was used as a basis for product development and the definition of requirements for the use of the product for the given purpose. This data also includes maintenance requirements to ensure the required efficiency and safety of the product.

**Disposal instructions:** For disposal, the product must be disposed of in accordance with the local valid environmental protection regulations in the country, where the product was used.

## **WARRANTY CONDITIONS:**

1. On this product is applied 24 months warranty from the date of sale. The warranty is applied for material and manufacturing defects.
2. The product has to be used entirely for the purpose it is designed for and the instructions in the user guide must be followed.
3. The product has to be properly used, maintained, cleaned and stored in accordance to the user guide instructions.
4. Do not pull down the identification tag. It has to be readable and undamaged for the whole warranty period.
5. The warranty case has to be claimed by the seller, where the product was bought. Properly filled "Warranty sheet" and the proof of purchase must be declared by the warranty application. Every defect should be claimed immediately
6. The claim procedure can be started when the device is delivered complete, in original packaging, carefully packed and secured against damage. The owner is committed to deliver the device clean in accordance to the general hygienic measures. The basic maintenance and cleaning cannot be expected or even required when claimed.
7. The warranty does not cover: failures caused by the overloading of the products, accident, forced damage, unauthorized changes made on the product and/or its accessories, natural wear and tear of parts, especially upholstery dirt and damage, wear of plastic parts, brakes, tyre profile and tyre puncture, etc.
8. The warranty is terminated whenever the following conditions arise: the user fails to follow the instruction

mentioned in this user guide and manufacturer's storage and maintenance suggested procedures, or whenever the product is serviced by any other party except of authorised service centres, or when the identification label is not stucked on the stroller and also in the warranty sheet.

9. The warranty period cannot be interchanged with using period, that means for the lifetime period when correct use and maintenance of the product in accordance to the user guide are fulfilled .
10. The producer nor the importer is not liable for any loses and injuries caused by manipulation and usage of the product in conflict with the "User Guide" instructions.

The manufacturer, PATRON Bohemia a.s., reserves the right to make changes to the products.

**NOTES:**

## **SCHEMA DESCRIPTION**

1. Handlebar
2. Backrest
3. Frontbar
4. Footrest
5. Chassis
6. Sidewalls
7. Coxal rest
8. Armrest
9. Left pedal to adjust the inclination of the seat
10. Right pedal for height adjustment of the seat

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## DISTRIBUTION CHAIN PARTICIPANT / IDENTIFICATION CHART

 <p>PATRON Bohemia a.s. Českolipská 3419, Mělník, CZ ( www.patron.eu )</p>	<p>Product identification label:</p>
<p>Manufacturer—product final inspection / name:</p>  <p>Signature:</p>	<p>Importer / EU-distributor (stamp):</p>
<p><u>Importer / EU-distributor :</u> (name, address, contact info)</p>	<p>Point of Sale / Service partner (stamp):</p>
<p><u>Point of Sale / Service partner :</u> (name, address, contact info)</p>	<p>Date of sale (day, month, year):                      Name (block letters), phone number, stamp and signature of the seller:</p>  <p>_____ . _____ . _____</p>