**INFORMATION FOR USE** ..... (en)

=> **medical device: rehab-stroller**

**GEBRAUCHANWEISUNG** ..... (de)

=> **Medizinprodukt: Reha-Kinderwagen**

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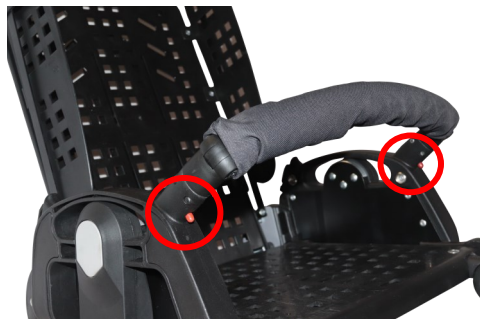
Obr.2

**INTRODUCTION:**

Dear customers, read this document carefully before using this product for the first time. Follow all safety instructions stated in this document. If you have any questions about the product and its proper use, contact professional staff or a service centre. Report any serious adverse event related to the product, or any deterioration in the user's health in connection with the use of the product, immediately to the official distributor of the product in your country or to the manufacturer of the product. For latest safety and/or recall information for this product, as well as the product declaration of conformity, please contact the official distributor of the product in your country or the product manufacturer. Keep this document in a safe place for further reference.

..... **WARNING, CAUTION:**

- ① **CAUTION! Any load attached to the handle (bag, ...) affects the stability of the stroller!**
- ② **The side padded frame covers must be used properly all the time!**
- ③ **Recommended tire air-pressure: 180 kPa**
- ⑥ **Footrest velcro lock must be properly tightened while using stroller!**

.....**maximum load.**

**WARNING! Keep fingers out of any moving parts when operating with the stroller.**

**WARNING! Never leave your child unattended! Even if the belts and brakes are in use.**

**WARNING! Check before use that all locking devices are properly locked and all safety equipments are properly used.**

**WARNING! Check before use that seat unit is properly locked in the chassis interface.**

**WARNING! The seat unit is not suitable for a child younger 6 months.**

**WARNING! This product must not be used while jogging or in-line skating.**

**WARNING! The seat unit must not be used in motor vehicles as the car seat.**

**ATTENTION! Read this manual carefully before the first use of the stroller, as well as keep it for further reference. Safety of a child might be in danger in case you will not follow the instructions and safety notices mentioned I the user guide.**

**PRODUCT - DESCRIPTION, FEATURES, IDENTIFICATION:**

You have received a product that is determined for versatile everyday usage outdoor, as well as indoor at home. Before using the product for the first time, get to know carefully how to use it , how to handle with it, how works all the product features, to avoid any possibility of injury. In particular, observe the following instructions:

- All users and / or their accompanying persons must be trained by qualified personnel how to operate with the product. Users and / or their accompanying persons are obliged to read and learn in particular about the residual risks of this product, in connection with the safety instructions given in this document.
- The product has been adapted to the individual needs of the user. Additional changes may only be made by qualified personnel.
- We recommend that you check the product customization once a year to ensure optimal product setup for users.

Especially for users with changing anatomy (eg body dimensions and weight) it is recommended to adjust at least once every six months.

- Your specific product may differ from the variants shown here. In particular, not all optional accessories described in this document are mounted on your product.
- The manufacturer reserves the right to make technical changes to the design described in this manual.

**Product features:**

This product is classified as a non-invasive therapeutic device of class 1. The product is intended for short-term and long-term use. The product is controlled by an accompanying person and is used to transport children with the help of another person. The product is intended for use on a solid surface in interiors and exteriors.

The product is used by children with temporary or permanent limitation of the ability to walk, unable to walk or keep standing. The product is suitable for users whose anatomy (eg body dimensions and weight) allows the use of this product for the given purpose and whose skin is intact. The product must only be used in conjunction with the accessories specified in the order form.

The manufacturer of this product assumes no responsibility for combinations with medical devices and / or accessories from other manufacturers. Only product combinations evaluated in terms of their efficiency and safety are permitted. This product and accessory combination must be permitted and stated in the product combination agreement.

Indications: mild to severe or complete restriction of the user's movement.

Contraindications: Absolute contraindications are not known. Relative contraindications - lack of physical or mental preconditions, significant surface damage of surface tissue (eg skin, etc.).

Accompanying phenomenon: Exceptionally, the following accompanying phenomena may occur when using the product - pain in the neck muscles, pain in the muscles and joints, circulatory disorders, risk of pressure sores. In case of these accompanying phenomena or problems, contact your doctor or therapist.

**Product identification:**

Information on the product label:

<b>GTIN</b>	Global trade identification number. According to GS1-standard = UDI-DI – Unique Device Identifier - Device Identifier. .
<b>SN</b>	Serial number. According to GS1-standard = UDI-PI – Unique Device Identifier - Production Identifier.
<b>REF</b>	Producer identification – product version
<b>Mod.:</b>	Product version / text
<b>Spec.:</b>	Product version / colour version
	Producer - address

Date of production: Serial number (first 3 digit) includes production date. („253“ = 253.-months from 01.01.2000 = January 2021)

**SAFETY AND PROTECTION:**

- The attendant must be familiar with the instructions of this manual.
- Professional medical advice regarding the suitability of this product must be sought before use.
- Always make sure, before departure, the child is comfortable, and that clothes and rugs etc, are tucked in, so they do not fall in the wheels.
- Ensure that child is clear of moving parts before making adjustments, to avoid any injury.
- When angle-positioning and adjusting the seat and backrest act accordingly to avoid sudden uncontrolled movements.
- Change of the seat direction must be done without the occupant and with empty seat unit.
- The buggy MUST NOT be used as a seat for a child while traveling in motor vehicle or bus if the vehicle is not equipped for that and if the buggy is not crash tested model according to the requirements of ISO 7176-19 or additionally of ANSI/RESNA WC-4:2012 Section 19 Annex A.
- Do not allow the child to climb unassisted into and out from the stroller.

- Always apply the parking brakes, when getting in and out the stroller.
- The stroller is not designed to be driven down the stairs.
- Ensure yourself that the fixing accessories, safety bar and other equipment is fitted and adjusted correctly.
- Do not stand on the footrest.
- The brakes must be engaged at all times when the stroller is not moving.
- Getting over the curb practice firstly with an empty stroller.
- The attendant must take precautions against the fire risk. In buildings, check the location of fire exits and ensure procedures are understood.
- Do not let children play with or hang on the stroller.
- During the drive keep full attention, to avoid any dangerous situation or collisions.
- Do not overload the stroller. Overloading of a chassis, seat, bag or basket causes reduction of stability of the stroller.
- Make the full attention, when manipulate with the stroller, not to injure the child or yourself.
- If it is possible, always use drive-up ramps or lift to overcome any access restrictions.
- Always lift the stroller in two persons.
- Never transport the stroller when you and/or assistance are not sure if you can transport the stroller with the child safely.
- When you drive the stroller on the road, follow the road traffic regulations.
- Do not use the stroller for any other purposes then it is designed for.
- The stroller is designed to carry only one child.
- Do not use any other than original PATRON accessories designed for this product.
- When the stroller is folded and unfolded, the user guide must be followed carefully, to avoid any damage on the construction what could affect the product stability.
- In case of apparent defect, which limits product's safe using (e.g. damage of chassis, part falls away or other optically visible defects, cracks etc.) it is necessary to stop using the product immediately and contact your local official distributor or service centre.
- Company PATRON does not bear any responsibility for the damages caused by using other than original PATRON accessories supplied by this company.
- Company PATRON does not bear any responsibility for the damages and accidents caused by inappropriate use of this product.

## **PREPARATION FOR USE, USE:**

### **Attaching and removing the seat from the chassis**

Attaching: Fix the seat onto "clip- and -go " interface from above , pressure on adapters on each side of the lock . Ensure proper signals by audible click. Pull the seat up , make sure it is correctly locked ( can not be ejected ) .

Removing : Press both buttons on the outside of the adapter harness , pulling upwards remove the seat from the chassis .

**WARNING !** Pay attention when carrying the seat !

### **Abductor block**

Seat the child into the stroller and find the appropriate place of the wedge. Take the child out of the stroller. Find the relevant fitting hole on the seat base. Place the wedge in the seat (the narrower part should aim to the backrest) and secure the wedge by a nut

### **Headwings**

Set the child into the stroller, to determine right height of headwings. Take the child out of the stroller. Put headwings in determined height onto the backrest, and secure it using straps with Velcro.

### **Removing the 5-point belt**

Open the 5-point belt buckle.

On the back of the backrest under the seat cushion, release the Velcro shoulder strap. Pull the strap from the back of the backrest and cover. At the end, pull the treadmill belt clip gradually through a trap in the seat and cover.

### **Installing a 5-point belt**

Pull the belt tensioner clip gradually through the cover, seat. Click the 5-point belt clips. Make sure it is not twisted. Take the shoulder strap, pull it through the cover. Loosen the velcro fastener on the backrest cover, and strap the shoulder straps through the trap and backrests of the seat. On the back side of the backrest under the cover, tie the shoulder straps together by the Velcro.

### **Assembling the undercarriage**

Lay compact undercarriage on the ground, put on all wheels. Remove protective covers of extensible locks on both ends of handle pipes. Stand between the rear wheels of the stroller on the side of the handle. Brake the undercarriage off by pressing the brake pipe to second marginal position, hold your foot slightly on the wheel's axis, grab the handle and pull it upwards, towards yourself, until you hear a click of the fuses. Make sure that the undercarriage is secured by light pushing the handle downwards (the undercarriage cannot be folded now).

### **Unfolding the undercarriage**

Brake the undercarriage off by pressing the brake pipe to second marginal position. With your foot on the pipe, grab

left and right swivel assembling controller on the handle pipe and rotate it inwards. To assembly the undercarriage to the marginal position, press with your forearm on the handle (until you feel a slight resistance from undercarriage chassis). Ensure with transport safety-catch. Assembled undercarriage can be braked off with your hand, if needed. Do not rest folded stroller by the extensible locks on handle pipes, the folding function could be damaged.

#### **Wheels**

**Mounting** - press the quick fastening mechanism of the wheel, put the wheel on the axis and release the mechanism. By pulling the wheel make sure, that the wheel is mounted correctly and cannot get loose.

**Releasing** - press the quick fastening mechanism of the wheel and remove it from the axis.

#### **Parking brake**

To apply the parking brake, press the plastic brake pedal. Please check the brakes regularly and keep good functionality by cleaning and greasing of the mechanism by silicone grease. ( plastic slider inside has to be loosely movable)

#### **Height of the handle**

The height of the handle can be set by pressing the black plastic articulations of the handle on both sides simultaneously and setting it to the desired position.

#### **Stroller operation:**

##### **Changing the position of the seat:**

The seat can be positioned in or against the direction of travel. Apply the brake on the stroller (stroller brake), take the child out off the seat. Grasp the seat firmly, simultaneously press both buttons on the outside of the adapter harness . Pulling upwards remove the seat from the chassis and turn in/against the driving direction. Place onto the "clip- and -go" interface from above , put on adapters on each side of the lock . Ensure proper placement with signals of audible click. Pulling the seat up , make sure it is correctly locked ( can not be ejected ) .

##### **Positioning of the seat:**

You can set four position of the backrest . When installing the backrest unlock mechanism positioning the backrest upwards , adjust it to the desired position and release mechanism with pressure on the backrest downwards you can make sure that the backrest is fixed properly (is not possible to change its position). When you lift the backrest, positioning is working automatically.

##### **Safety Bar:**

To fit the safety bar put both ends of safety bar in seat side tubes at once (pic.2). To take out the safety bar is ensured by the spring locking on both sides., press press-pin release and lift the safety bar up.

##### **Footrest height adjustment:**

Hold the footrest. Loose the footrest height adjusting bolts. Set the position of footrest and tighten properly both adjusting bolts. Turn adjusting bolts small handles into the safest position by pulling it and rotating.

##### **Footrest set reclining:**

Press both press buttons (pic.1) on footrest hinge simultaneously and adjust the angle position of footrest. Free both buttons and set the position of footrest to allow the hinges to be locked.

##### **Getting over the curbs and low obstacles:**

*Down* – hold the handlebar tightly; push the stroller upright with the front wheels to edge of the curb, press a little handlebar down to lift a bit front wheels and then carefully ride off the curb on the rear wheels.

*Up* - push the stroller upright to the curb, press a little handlebar down to lift front wheels to desired height and ride on to the walkway. Hold the handlebar tightly to keep stroller's side stability. Pull the handlebar a little up to lift the rear wheels to desired height and ride on to the walkway. In any case do not bump with the front wheels to the curb!

##### **Carrying stroller:**

Use ramps and lifts if possible. If you have to get over the stairs or other obstacles, the stroller must be carried by 2 persons at least. Never do that if you or your assistant are not sure that you can carry the stroller with child safely. During the carriage child must not have the hands out of stroller.

**WARNINGS!!** NEVER lift the stroller up by holding the grab bar, foot-rest, seat unit, carrycot or hood of the seat/ carrycot!!

##### **Down from the stairs:**

Hold the handlebar tightly; push the stroller upright with the front wheels to the top edge of the stairs. Your assistant grabs the chassis above the front wheels. Both of you lift up the stroller, carefully carry that down and set back again on all wheels.

##### **Up to the stairs:**

Hold the handlebar tightly; bring upright the stroller backing up to the lower step of the stairs. Your assistant grabs the chassis above the front wheels. Both of you lift up the stroller and carry upstairs so you are going first and your assistant second. Set the stroller back on all wheels in safe distance behind top edge of stairs.

## **MAINTENANCE, PRODUCT DISPOSAL:**

The stroller has been designed to the high standard and with correct usage and maintenance will provide many years of trouble free service.

- make sure that you are familiar with the operation of the stroller. It should always open and fold easily. If it does not, do not force the mechanism - stop and read the instructions
- regularly inspect handle folding release, safety bar fittings, locking devices, catches, footrest, joints fixings and handle release safety clip to ensure they are secure, clean and in full working order
- brakes, wheels, tyres and upholstery are subjects to wear and should be replaced if necessary by a service centre.
- If you have inflatable tires on the stroller, check its inflation regularly. Allowed tire unevenness on the perimeter of tire is +/- 2 mm
- if the stroller is damaged or in need of any repair, only authorised service centres are allowed to provide these services.
- only spare parts supplied by company PATRON has to be used for repairs.
- should parts of your stroller become stiff or difficult to operate, ensure that dirt is not the cause, apply a light application of a spray lubricant e. g. WD 40 (do not use oil or grease). Ensure that the lubricant penetrates between the components but do not overspray as this may encourage the accumulation of dirt. Wipe off any spray excess. Regular lubrication of moving parts will extend the life of your stroller and make opening and folding easier.
- it is recommended to use rain cover when it rains (it is not standard equipment of the strollers)
- do not expose the stroller to the long-acting intensive solar radiation
- textile components of stroller and/or accessory must be used and stored in the way that the transparent plastic parts are not folded or turned tightly. Those can be damaged if the temperature is 5°C below zero.
- wheels, plastic and metal parts may be sponged clean with warm water and mild detergent.
- the seat fabric must be hand washed only at maximum temperature of 40°C, do not use laundry dryer.
- never clean with abrasive, ammonia based, bleaches and alcohol based cleaning compounds.
- after exposure to damp conditions, dry the product off with a soft cloth and leave fully open in warm and dry environment.
- store your stroller in a dry, safe place with recommended temperature 5-20°C.
- it is recommended that the stroller is checked by an approved service centre every 12 months
- We recommend that after 24 months visit a service department.
- Check and tighten the screw connections regularly.
- If there are used control cables (brakes, telescopes, tilt), check it regularly and, if necessary, set up and oil them.

Product lifetime: The expected product lifetime is 5 years. The expected product lifetime was used as a basis for product development and the definition of requirements for the use of the product for the given purpose. This data also includes maintenance requirements to ensure the required efficiency and safety of the product.

Disposal instructions: For disposal, the product must be disposed of in accordance with the local valid environmental protection regulations in the country, where the product was used.

## **WARRANTY CONDITIONS:**

1. On this product is applied 24 months warranty from the date of sale. The warranty is applied for material and manufacturing defects.
2. The product has to be used entirely for the purpose it is designed for and the instructions in the user guide must be followed.
3. The product has to be properly used, maintained, cleaned and stored in accordance to the user guide instructions.
4. Do not pull down the identification tag. It has to be readable and undamaged for the whole warranty period.
5. The warranty case has to be claimed by the seller, where the product was bought. Properly filled "Warranty sheet" and the proof of purchase must be declared by the warranty application. Every defect should be claimed immediately
6. The claim procedure can be started when the device is delivered complete, in original packaging, carefully packed and secured against damage. The owner is committed to deliver the device clean in accordance to the general hygienic measures. The basic maintenance and cleaning cannot be expected or even required when claimed.
7. The warranty does not cover: failures caused by the overloading of the products, accident, forced damage, unauthorized changes made on the product and/or its accessories, natural wear and tear of parts, especially upholstery dirt and damage, wear of plastic parts, brakes, tyre profile and tyre puncture, etc.
8. The warranty is terminated whenever the following conditions arise: the user fails to follow the instruction mentioned in this user guide and manufacturer's storage and maintenance suggested procedures, or whenever the product is serviced by any other party except of authorised service centres, or when the identification label is not stucked on the stroller and also in the warranty sheet.
9. The warranty period cannot be interchanged with using period, that means for the lifetime period when correct use and maintenance of the product in accordance to the user guide are fulfilled .
10. The producer nor the importer is not liable for any losses and injuries caused by manipulation and usage of the product in conflict with the "User Guide" instructions.

The manufacturer, PATRON Bohemia a.s., reserves the right to make changes to the products.

**NOTES:**

## SCHEMA DESCRIPTION

1. Handle
2. Handle hight adjustement hinge
3. Transport safety catch
4. Swivel assembling controller
5. Folding lock
6. Rear wheels
7. Wheel quick release button
8. Brake
9. „clip-A-go“ interface for seat unit/carrycot/baby car seat adapter
10. Brake foot pedal
11. Front wheels
12. Basket
13. Lock
14. footrest position adjusting bolt
15. footrest height adjustment
16. Safety Bar
17. Headwings

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## DISTRIBUTION CHAIN PARTICIPANT / IDENTIFICATION CHART

 <p>PATRON Bohemia a.s. Českolipská 3419, Mělník, CZ ( www.patron.eu )</p>	<p>Product identification label:</p>
<p>Manufacturer—product final inspection / name:</p>  <p>Signature:</p>	<p>Importer / EU-distributor (stamp):</p>
<p><u>Importer / EU-distributor :</u> (name, address, contact info)</p>	<p>Point of Sale / Service partner (stamp):</p>
<p><u>Point of Sale / Service partner :</u> (name, address, contact info)</p>	<p>Date of sale (day, month, year):                      Name (block letters), phone number, stamp and signature of the seller:</p>  <p>_____ . _____ . _____</p>